


To store fresh breastmilk, connect a freezer bag to the pump body and express directly into the bag. Snap rings to attach freezer bags to the pump body are supplied with the freezer bags.

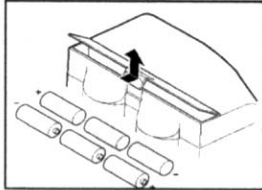

The Lactaline Personal may also operate with 6AA alkaline batteries. Open the battery cover (6) and insert the batteries into the compartment, ensure polarity (+) and (-) is correct. Close the battery cover by gently pressing the lid until it snaps into place.

For convenience, the Lactaline Personal may be used in the car by connecting a special adaptor, also available from Amber Medical, to the car cigarette lighter.

After use, wipe over the pump housing with a soft cloth dampened with water and a mild detergent.

NEVER IMMERSE THE PUMP IN WATER.

Technical data: the Lactaline Personal breast is a medical device which complies with guideline 93/42 and UL/CSA.

Operating conditions: +10—+40°C / 10—90% r.h.

Storage Conditions: -40—+50°C / 10 —100%r.h.

Vacuum range: 0—330mbar.

Cycles: 30—60 cycles per minute.

Cleaning the milk collection set: Always dismantle the collection set prior to cleaning. As tubing, dual kit connector and snap-on lid do not come into contact with mother's milk during the collection cycle, in the event of spillage onto to these parts during disassembly, wipe with mild detergent, rinse and dry immediately – **NEVER IMMERSE THESE PARTS IN WATER.** Wash other parts in a mild detergent and rinse clean with a stream of cold water.

Sterilising the milk collection set: Before reassembly, it is recommended that the set, **except for THE SNAP-ON-LID, DUAL KIT CONNECTOR OR THE SILICONE TUBING, is** sterilised using one of the following methods.

- . Boiling water: for 10-20 minutes
- . Cold water: follow the instructions stated by the manufacturer of the sterilising chemicals.
- . Steam sterilisation: carry out in accordance with the instructions set by the manufacturer of the steam steriliser.

Ensure that all parts are dry before reassembling the set.

THIS PRODUCT IS NOT SUITABLE FOR MICROWAVE STERILISATION